progetti[®]
Medical Equipment Solutions

RESCUE Life 9

Think different, **be** different

Defibrillator

Technology for **diagnosis** and **emergency**

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Usability

Close to the needs of the modern emergency medicine, Rescue Life 9 guarantees a user friendly interface, an impact-proof 8.4" LCD display and powerful batteries that ensure up to 9 hours continuous functioning. Different product configurations are available, from the standard one to the fully equipped device with all optionals included. Periodically-run system tests and shock tests and possibility to print detailed reports ensure readiness and top operating conditions.

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Design

The Rescue Life 9 concept comes from a study of an ergonomic, attractive, high performance defibrillator that expresses Progetti Medical's vision for the future of emergency. It was born from expert Italian designers reflecting the **traditions of elegance and comfort**. Thanks to the lightweight and transport bag, Rescue Life 9 is perfect for hard challenges. Quality, beauty, and safety are the result of our expertise, applied to our entire product line.

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Advanced Technology

Rescue Life 9 is the new flagship product resulting from the continuous research and development carried out for more than 30 years by Progetti Medical. The device generates a **Biphasic Truncated Exponential (BTE) defibrillation waveform**, compensated with the patient's impedance. If the impedance is detected incorrectly, the device warns the user with a message on the screen.

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A different point of view

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Display

Bright high-resolution **TFT LCD color 8.4"** display allowing the simultaneous real-time visualisation of 3-lead, 7-lead, and 12-lead ECG and the plethysmograph waveform with SpO₂ sensor connected. ECG leads error message is displayed when the cable is disconnected. Customizable ECG traces in the setup menu. **Alphanumeric keyboard** to input patient data.

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Fast Connection

Users can choose between ergonomic reusable paddles for manual defibrillation or disposable pads for the AED mode. Both types are equipped with a single fast lock connection system for reliable rescue operations. Single selector for direct access to the menu and different energy levels. **Fast responding**: very short starting time and ready to defibrillate in less than 5 seconds.

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Smart User Support

Dedicated area on display for visual messages, using different colors for each parameter. Clear voice prompts and dedicated buttons provide quick and intuitive access to the main functions. The **double battery pack** assures prolonged working time.

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High performance ECG analysis

Rescue Life 9 can filter the ECG signal automatically thanks to two functioning modes: **Diagnostic**: ECG signal acquisition recommended to qualified healthcare personnel for ECG traces interpretation and diagnosis. **Monitoring**: digitally filtered ECG signal, removing most of noises and artifacts caused by patient motion or other physiological processes. Through **3-5-10-lead** ECG cables, it is possible to **view and print the ECG traces** in real time, supporting clinicians on suspected arrhythmias. More than 30 selectable energy levels.

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Memory and Data Management

Rescue Life 9 can register and store up to **10 hours of events and ECG traces** and each record holds the 12-lead ECG data. The recording is available in Advisory mode and in AED mode. Each time the analysis starts, the recording will start automatically. Records can be printed or transferred through a USB pen and then analyzed on PC using the **PG Data Manager Software**.

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AED & Advisory Mode

Step 1-2-3 guide supports the operator during use in AED mode. **Voice prompts and visual messages** can guide the procedure of CPR and shock release. The Advisory mode analyzes the ECG rhythm automatically and continuously through the ECG cable or the pads connected to the patients and advises whether the shock is necessary.

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Alarm Mode

Configurable alarms for every single function. In case of exceeding the set limit, an alarm message will appear in the dedicated area on the screen. The automatic printing function allows to have a report automatically, before and after each shock release, including all 12-leads ECG.

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A different point of view

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Configuration and Optionals

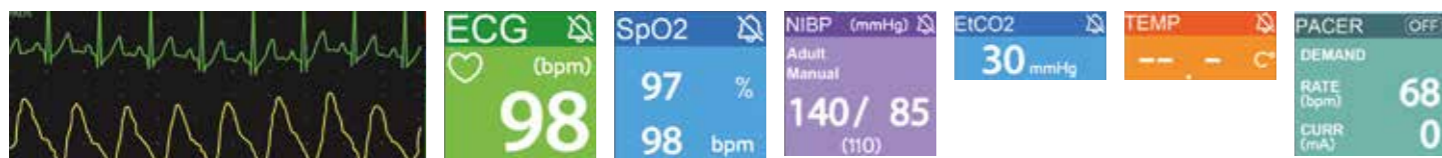
The Rescue Life 9 is a versatile and fully customizable defibrillator. In its standard configuration, it offers **three operating modes** -AED, Advisory, and Manual- and includes a **synchronization function** for **controlled shock delivery**. It is supplied with reusable adult and pediatric rigid paddles, a 7-lead ECG cable, an adapter cable, and a set of disposable adult pads.

The Rescue Life 9 can be easily configured with additional modules and accessories to meet specific clinical needs. Available options include a **3 or 12-lead ECG cable**, a **pacemaker module**-both fixed and on-demand-a **SpO₂ module** with an adult sensor, an **NIBP module** with an adult cuff, a **temperature module** with an adult probe, and an **EtCO₂ module** with an adult disposable adapter.

To complete the package, there is a wide range of dedicated accessories, such as the ergonomic carrying bag available in both rear and side versions, a second battery to increase the device's autonomy, the 12V adapter cable, and the ambulance mounting bracket, all the pediatric accessories related to the available modules, and much more.

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Technical specifications

GENERAL SPECIFICATIONS

Size & weight • 335 x 225 x 360 mm • 6,5 Kg with two batteries
Power supply • 100-240 V_{AC}, 50-60 Hz • Rechargeable batteries: 14.4 V_{DC}, 3.4 Ah Lithium-ion
Battery capacity • 4.5 hours with one battery; 9 hours with two batteries
Battery charge time • <2 hours with one battery; <4 hours with two batteries
Shocks • 200 shocks with one new fully-charged battery at 230 J or 500 shocks with two new fully-charged batteries at 230 J
Software • Multilingual prompts and texts
Memory • Internal memory for ECG data, events up to 10 hours
Applied parts type • for ECG: CF defibrillation proof; for defibrillation and other modules: BF defibrillation proof
Ingress protection • IP54

ENVIRONMENTAL CONDITIONS

Temperature • Operational -10° ~ 50° C • Storage -40° ~ 70° C
Humidity • Operational 15 ~ 90 %

DEFIBRILLATION

Waveform • Biphasic Truncated Exponential (BTE)
Impedance • Compensated, range 25 - 200 Ohm
Charge time • < 7 s for 230 J with fully charged batteries

AED MODE

Energy • Preset energy at 200 J; reduced to 50 J with pediatric disposable pads
Protocol • ERC 2021 Guidelines – CPR with voice prompts and guiding messages
Shockable rhythms • VF with amplitude > 0.15 mV and VT with rhythm >150 bpm
Sensitivity • Shockable rhythm-VF > 95%, Shockable rhythm-VT > 75%
Specificity • Normal sinusoidal rhythm > 99% • Asystole and other non-shockable rhythms > 99%

MANUAL MODE

Energy range • 1 - 230 J (from 1-10 J with 1-J adjustments; from 10 to 230 J with 10-J adjustments)
Keys • Multifunction knob. Charge and discharge buttons directly on the front panel for hands-free defibrillation (with disposable multifunction electrodes)
Paddles • Adult and pediatric reusable standard electrodes with charge/shock button • Adult and pediatric disposable multifunction electrodes
Defibrillation mode • Shock in sync or not sync with the ECG

DISPLAY

Type • High resolution TFT color LCD 8.4" (800X600dots)
Content • Working information, ECG waveform and value, HR, SpO₂, EtCO₂, TEMP, and NIBP values, information on filters, alarms and battery/ies charge status
Display mode • 3-, 7-, 12-lead ECG can be visualized on screen

ECG

Bandwidth • 0.05 - 120 Hz (-3 dB) in diagnostic mode • 0.6 - 40 Hz (-3 dB) in monitoring mode
CMRR • > 98 dB
Input impedance • 50 Mohm
ECG cable • 3-lead ECG cable (selectable I, II or III) - optional
• 7-lead ECG cable (I, II, III, aVL, aVF, aVR, V) - standard
• 12-lead ECG cable (I, II, III, aVL, aVF, aVR, V1.....V6) - optional
Gain • 2,5, 5, 10, 20, 40 mm/mV
Speed • 5, 10, 25, 50 mm/s
Filters • 50-60 Hz, EMG filter, baseline
HR range • 20 - 300 bpm (±5% or ±3 bpm, highest value), digital readout on screen
Alarm • Minimum HR 20 bpm and maximum HR 250 bpm, customizable values

SpO₂ (OPTIONAL)

SpO₂ range • 0 - 100%
HR range • 30 - 250 bpm
Accuracy • 70 - 100% ±2% for adults with finger clip sensor
Alarm • Adjustable, min 50%

NIBP (OPTIONAL)

Technique • Oscillometric
Accuracy • EN IEC 80601-2-30
Patient application • Adult/paediatric/neonatal
Systolic range • Adult: 40-260 mmHg, paediatric: 40-160 mmHg, neonatal: 40-130 mmHg
MAP range • Adult: 26-220 mmHg, paediatric: 26-133 mmHg, neonatal: 26-110 mmHg
Diastolic range • Adult: 20-200 mmHg, paediatric: 20-120 mmHg, neonatal: 20-100 mmHg
PR range • 30-220 bpm
PR accuracy • ±2% or ±3 bpm, the highest value
Operating modes • Manual, Long Term Automatic, STAT, Service
Intervals • In Automatic mode: 1, 2, 3, 4, 5, 10, 15, 30, 60, and 90 minutes

PRINTER

Type • Integrated 3-channel thermal printer for ECG tracing, event information, and values of parameters, such as HR, SpO₂, NIBP, EtCO₂, and TEMP
Printing speed • 5, 10, 25, 50 mm/s
Paper width • 58 mm
Working mode • Manual, automatic (12" pre- and post-shock recording)

EXTERNAL PACEMAKER (OPTIONAL)

Pulse shape • Rectangular wave
Operating mode • Permanent, on demand
Impulse rate • 30 - 170 bpm, adjustable in steps of 1 bpm
Impulse duration • 20, 30, 40 ms
Pulse current • 0 - 200 mA, adjustable in steps of 5 mA

EtCO₂ (OPTIONAL)

Mainstream technology for both intubated and non-intubated patients
Measurement range • 0 to 150 mmHg, accuracy ±2%
Respiratory rate • 0 to 150 br/min, ± 1 br/min

TEMPERATURE (OPTIONAL)

Probe type • YSI400 compatible
Range • 0 to 50 °C
Resolution • ±0.1 °C

Standard • IEC/EN 60601-2-4; IEC/EN 60601-1; IEC/EN 60601-1-2; IEC/EN 60601-1-12; EN 1789
Marking • EC according to Dir. 93/42/EEC

SPECIFICATIONS AND CONTENTS OF THIS DOCUMENT MAY BE SUBJECT TO CHANGE WITHOUT NOTICE

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For over 30 years we have been developing products aimed at **prevention** and **treatment** of heart diseases.

Thanks to our international network and presence in over 70 countries worldwide, we are committed to providing **the best Italian technology in the medical field**.

For this reason, several companies, institutions, associations, sports clubs, enterprises, and many others choose Progetti Medical daily.

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PROGETTI S.r.l.